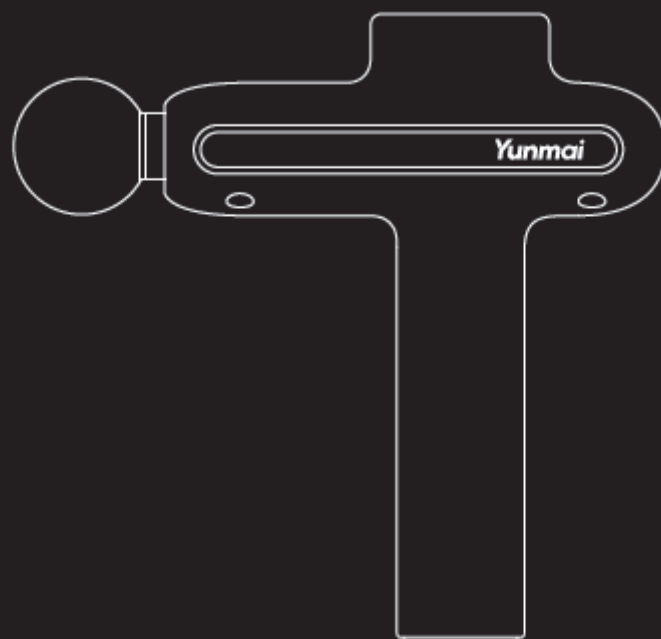
YUNMAI Fascia Massage Gun Slim Elegant

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The product user guide contains product features, how to use, and the operating procedure. Read the user manual carefully to get the best experience and avoid unnecessary damage. Keep this manual for future reference. If you have any questions or comments about the device, please contact the customer line.

✉ www.alza.co.uk/kontakt

☎ +44 (0)203 514 4411

Importer Alza.cz a.s., Jankovcova 1522/53, Holešovice, 170 00 Praha 7, www.alza.cz

Key Information

To reduce risks of malfunction, electric shock, injury, fire, death and damage to equipment, please read all statements and warnings in this manual carefully before use.

Statement

Those having any one of the following circumstances shall consult the doctor before using.

1. Pregnant women
2. Pacemaker wearer
3. Undergone surgery recently
4. Abnormalities in spines, joints or bones
5. Complications caused by other diseases (such as neuropathy or retinal damage)

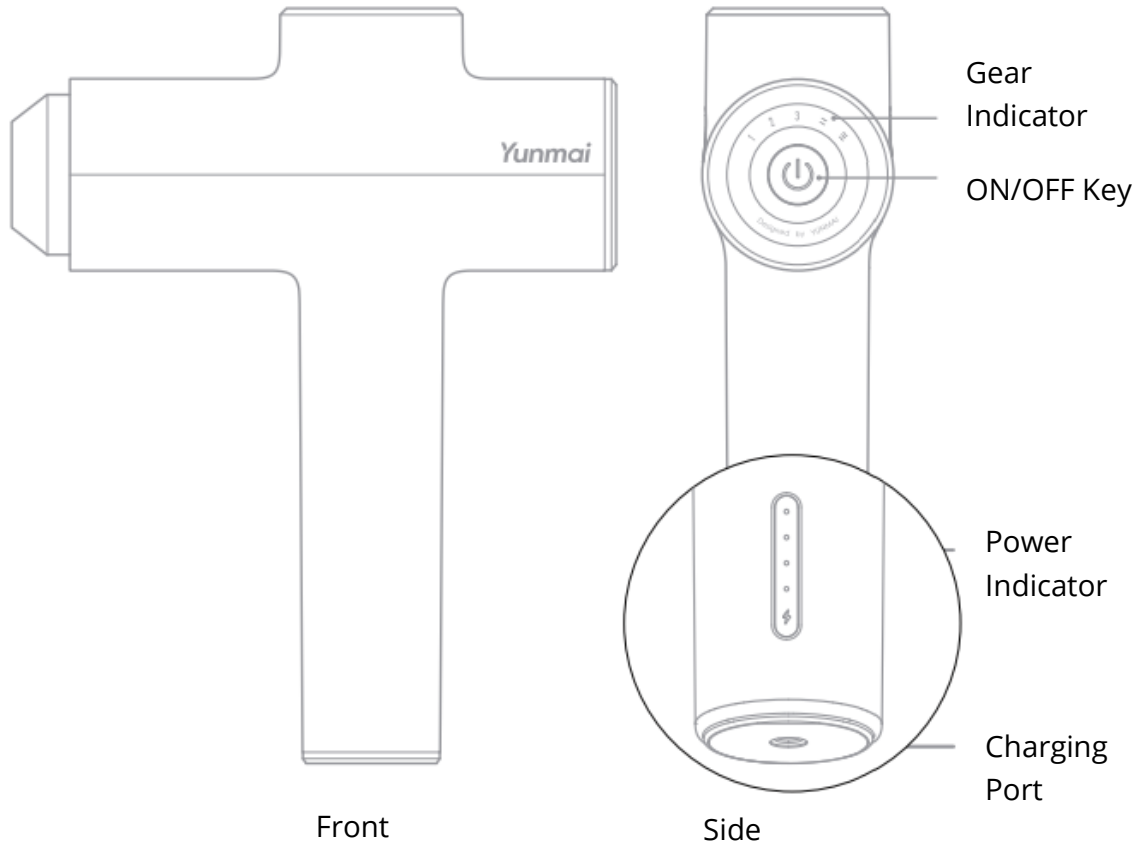
Current research focuses on the influence of vibrational training on special medical conditions and may reduce the scope for the above-mentioned contraindications. Practical experience has shown that the combination of treatment and vibrational training also serves as a type of therapy in many circumstances. However, such an approach should be recommended by physicians, experts or therapists or be conducted in their presence.

Precautions

01. Unplug the charger after the completion of charging.
02. Do not use the product when it is charged.
03. Please use the product according to the instructions.
04. Before each use, carefully check whether the product can be used normally according to the operation methods stipulated in the instructions.

Product Introduction

Thank you for choosing Yunmai Massage Gun Slim Elegant

Massage head

Round Head
Mainly used for large muscle group

U-shaped Head
Mainly used for the neck, both sides of spines and Achilles tendons

Flat Head
Mainly used to relax and shape muscle parts

Bullet head
Mainly used to impact deep tissues

Use Method

1) Gear indicator: 3 speeds for standard mode, 2 speeds for massage mode

2) ON / OFF key:

- Long press the ON / OFF button for 2 seconds until activate both of "power " and "mode" indicator lights. There are 5 modes for relaxing your muscle. You can switch between standard mode 1-3 and massage mode 1-2 by a short-pressing.
- Massage Gun will enter the standby-mode with a short-pressing after massage mode 2.
- Long press to turn off in any mode.
- In standby-mode, without operation for 15s, the product will turn off.
- The product will enter the intelligent power-off protection mode after continuous use of 10min. Restart the product if you need to use it.

3) ON / OFF key:

1) When the indicator lights are all on, the battery power is 100%."

2) When three indicator lights are on, the battery power is 65%-80%."

3) When two indicator lights are on, the battery power is 40%-60%."

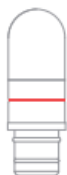
4) When one indicator light is on, the battery power is 20%-35%. The battery is low and needs to be charged if only one light is on. (When the battery is on charge, the indicator lights flash regularly. If all lights are on, it is fully charged)

Note: When starting the massage mode, the motor speed will be under sinusoidal variation. This process will produce the wind sound of acceleration and deceleration, which is normal.

Massage Head Assembling

Round head

Bullet head

Flat head

U-shaped head

Notes:

The red solid line in the figure is the line for alignment, which is bright black in the actual product.

When assembling or replacing the massage head, please insert the massage head directly into the mouth of the massage gun until the line for alignment coincides with the mouth (as shown in the figure); when replacing the massage head, pull out the massage head directly.


**Introduction of the removable gel**

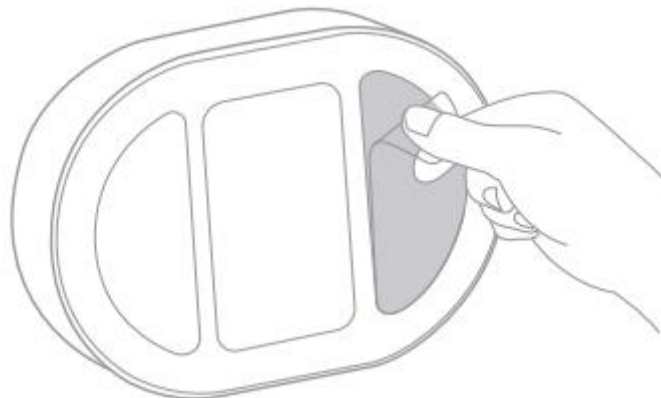Under the charging base, there are two gels. User can remove the plastic sheet on it and stick the charging base on a clean surface. (The gel is removable, which can be washed and used repeatedly.)

Note:

Do not place the charging base on the ramp or the edge to prevent the massage gun falling off and damaged.

Do not stick the charging base on the dusty place, otherwise the stickiness of the gel will be affected.

**Clean the removable gel**

1. When the gel is not sticky, you can wipe it with a little water and dry in the shade.
2. Do not wipe it with paper, towel or any other object that are easy to drop crumb.
3. Do not wash the charging base to avoid short circuit.

Basic Parameters

Name: YUNMAI Fascia Massage Gun SE	Type: MV-FG-0308
Net weight: 0.84kg	Input voltage: 100-240V-50/60Hz
Interface type: DC, 19V / 1A	Battery capacity: 2900mAh"
Intensity gears: 5 gears for adjustment	Charging time: About 4h
Power supply mode:	Applicable environment: 0-40°C
Size: 179 x 60 x 236mm	Applicable age: Adults above the age of 18 (except pregnant women)

Frequently Asked Questions

Start-up failure

High temperature:

When used in the environment above 40°C, the fascia gun may trigger overheating protection, causing normal start-up failure of the fascia gun.

Solution:

Place the fascia gun in an environment below 40°C for 2h before starting it up for use.

2. Battery capacity is " 0 "

When the battery is out of power, if the user does not charge it for a long time, the battery will initiate low-battery protection, collectively referred to as over discharge protection.

The fascia gun cannot work normally if it is started at this time.

Solution:

Keep the fascia gun off, then connect the charger (power adapter), and the fascia gun will automatically enter the charging state. When it is fully charged, unplug the charger, and the fascia gun can work normally.

Charging failure

1. High fascia gun temperature:

When the temperature of the fascia gun body is too high, it cannot be charged when it is connected to the charger.

Solution:

Place the fascia gun in an environment below 40°C for 2h before charging.

2. High charger temperature:

When the fascia gun is charged in the environment above 40°C, the power adapter may trigger overheating protection, thus causing normal charging failure of the fascia gun.

Solution:

Place the fascia gun in an environment below 40°C for 2h before charging.

Troubleshooting of Fascia Gun Operation

Fault description	Troubleshooting
1.The massage head slides out and falls	Check whether the massage head is installed in place (see page 5 of the instructions for specific operation)
2.The product cannot be started	Check whether the power is sufficient (if the product has not been used for a long time, please charge it for more than 8 hours)
3. The product cannot be charged	Check whether the charging connector and power supply are plugged in place
4. The product stops suddenly after 10min of continuous operation	This phenomenon is normal as it is caused by the setting for long-term operation protection, which switches from operation to standby mode. And the product can be restarted through short pressing (see page 4 of the instructions for specific operation)
5. The product stops working after the massage head encounters the phenomenon of locked rotor	This phenomenon is normal as it is caused by the setting for abnormal operation protection, which switches from operation to standby mode. And the product can be restarted through short pressing (see page 4 of the instructions for specific operation)

Reasonable use of the product can effectively extend its service life

Suggestions:

1. Do not throw or cast the product during use.
2. Keep the maximum continuous service time lower than 30min.
3. Please charge the product every other month if it is not used for a long time.

Recycling

Do not discard this device together with general household waste when it is at the end of its service life.

Place it in the government-appointed recycling bin or send it back to after-sales service centre where workers will dispose of it in an environmentally friendly manner.

Maintenance, Cleaning, Storage, Transportation

1. Clean the device with a damp towel and wipe dry with a soft cloth.
2. Place the device inside the packaging box provided during storage or travelling.
3. This device should be stored in an area that is dry, well ventilated, and non-corrosive. Device should be placed no less than 250mm away from the floor, and the maximum number of layers on top of it should not be more than 5.
4. This device should be packed and stored properly if it is to be left unused for a long period of time. In such circumstances, the device should be recharged once every 6 months.
5. This device can be transported by car, train, sea, or air.
6. During transportation, this device should be gently handled and well-secured in an upright position to avoid damage due to strong vibration, collision, rolling or falling. Water-proofing measures should also be in place.

Name of parts	Hazardous and noxious substances					
	(Pb)	(Hg)	(Cd)	(Cr(VI))	(PBB)	(PBDE)
Shell	0	0	0	0	0	0
Circuit board assembly	x	0	0	0	0	0
Metal parts	0	0	0	0	0	0
Rubber parts	0	0	0	0	0	0
Adapter	x	0	0	0	0	0
Connecting line	0	0	0	0	0	0
Handlebar grip	0	0	0	0	0	0

This table has been compiled in accordance with 5J/T 11364

X : Circuit board assembly consists of parts of printed circuit board

machine, such as resistors, capacitors, integrated circuits, connectors.

0: It means that the content of hazardous substances contained in homogeneous materials of this device is within the limit specified in GB/T26572.

X : It means that the content of hazardous substances contained in some homogeneous material of this device is beyond the limit specified in GB/T26572.

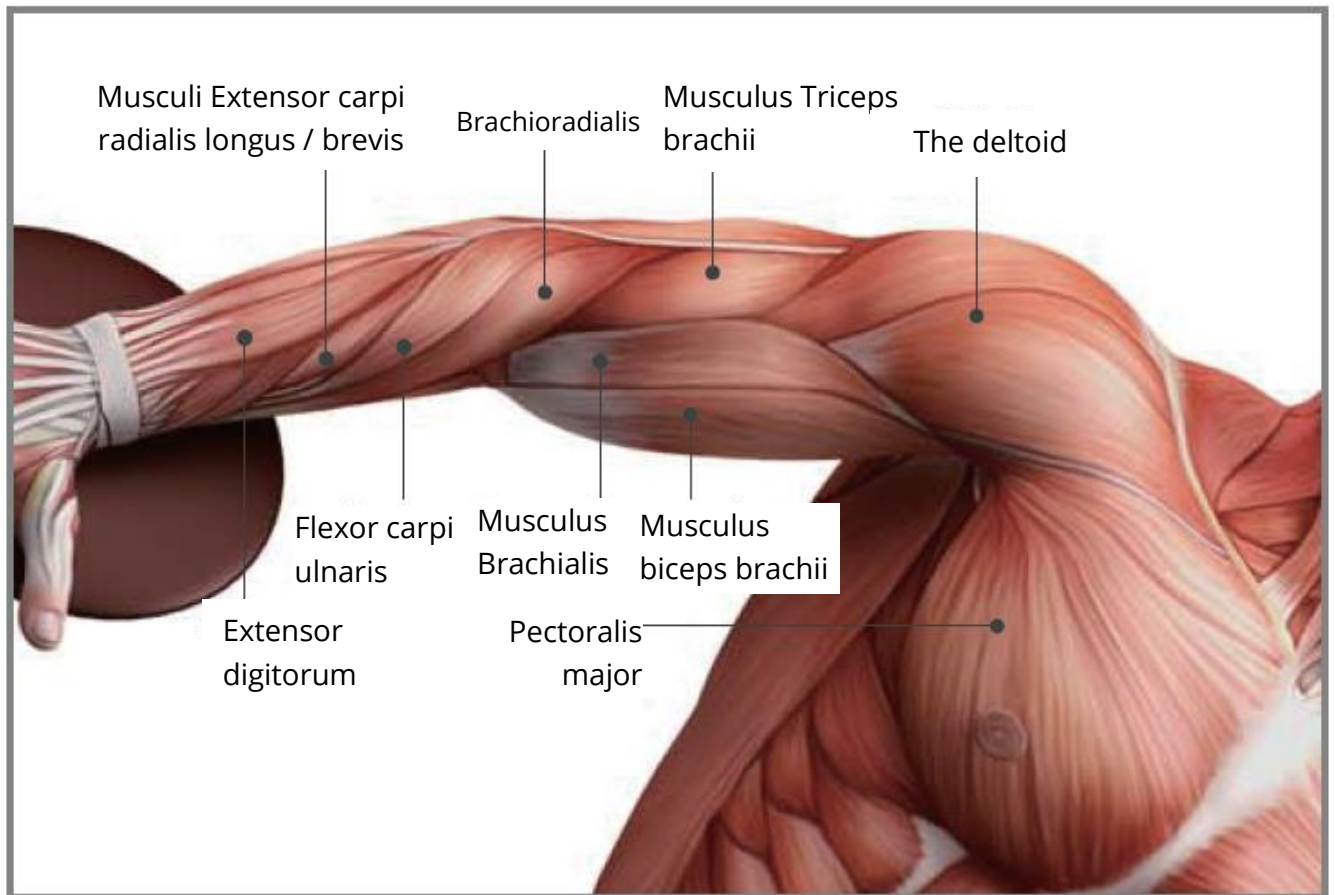
Instructions on environmental service life

Hazardous substances or elements contained in this device will not leak if this device is used under the conditions stated in the manual. The device can be used for a period of 10 years without causing any environmental pollution or severe harm to the human body or property


Massage Position

Arms


Position	Duration (s)	Frequency
Brachioradialis muscle	10 – 15 s	x3 times
Flexor carpi radialis	10 – 15 s	x3 times
Palmaris longus	12 – 15 s	x3 times
Flexor carpi ulnaris	10 – 15 s	x3 times
Extensor digitorum	05 – 10 s	x2 twice
Extensor carpi longus radialis	05 – 10 s	x2 twice
Extensor carpi radialis brevis	05 – 10 s	x2 twice
The biceps brachii	12 – 20 s	x3 times
Brachialis muscle	12 – 20 s	x3 times
The deltoid	12 – 20 s	x3 times
Pectoralis major	12 – 20 s	x3 times

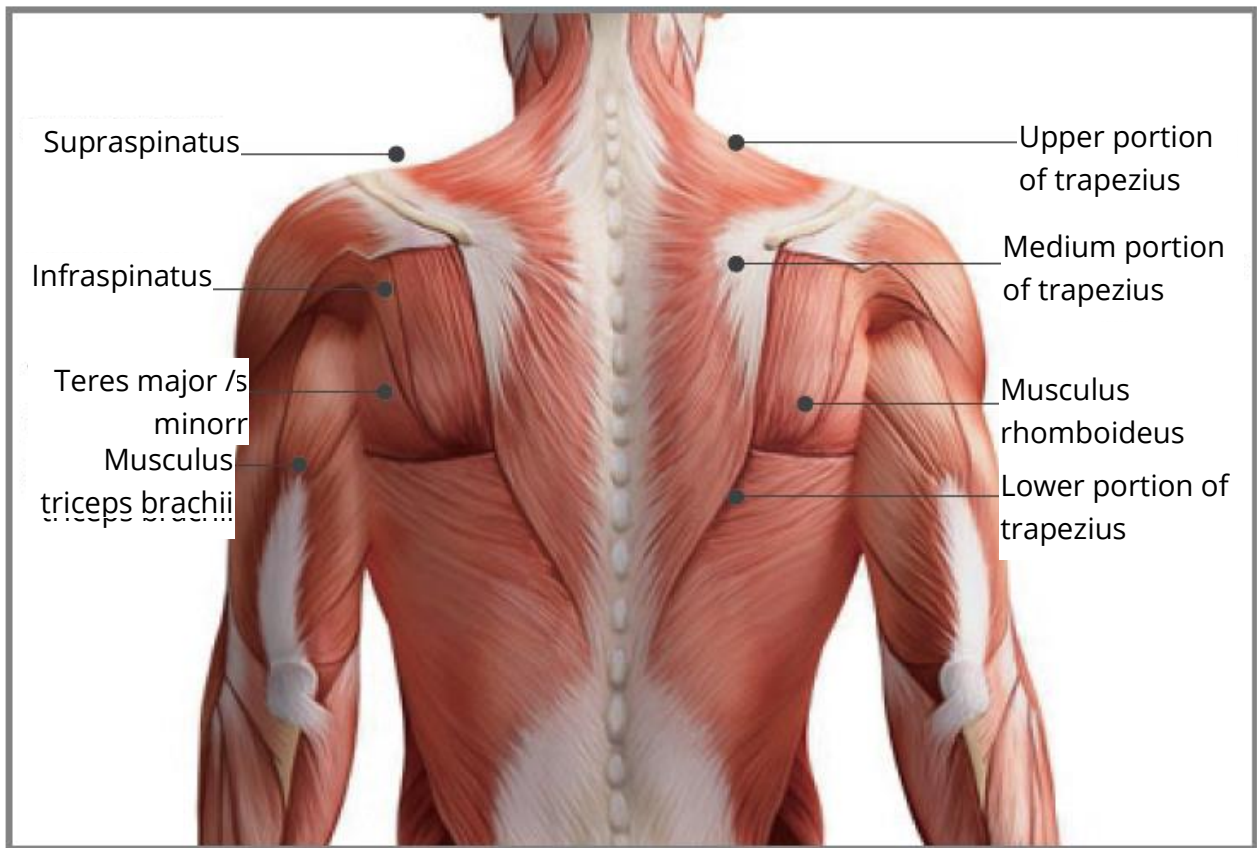Shoulder

Position	Duration (s)	Frequency
Triceps brachii	15 – 20 s	x3 times
Musculus Rhomboideus	20 – 30 s	x3 times
Upper portion of trapezius muscle	15 – 20 s	x3 times
Middle portion of trapezius muscle	15 – 20 s	x3 times
Lower portion of trapezius	15 – 20 s	x3 times
Supraspinatus muscle	10 – 15 s	x3 times
Elevator scapulae	10 – 12 s	x2 twice
Teres major muscle	10 – 12 s	x2 twice
Teres minor muscle	10 – 12 s	x2 twice
Infraspinatus muscle	10 – 12 s	x2 twice

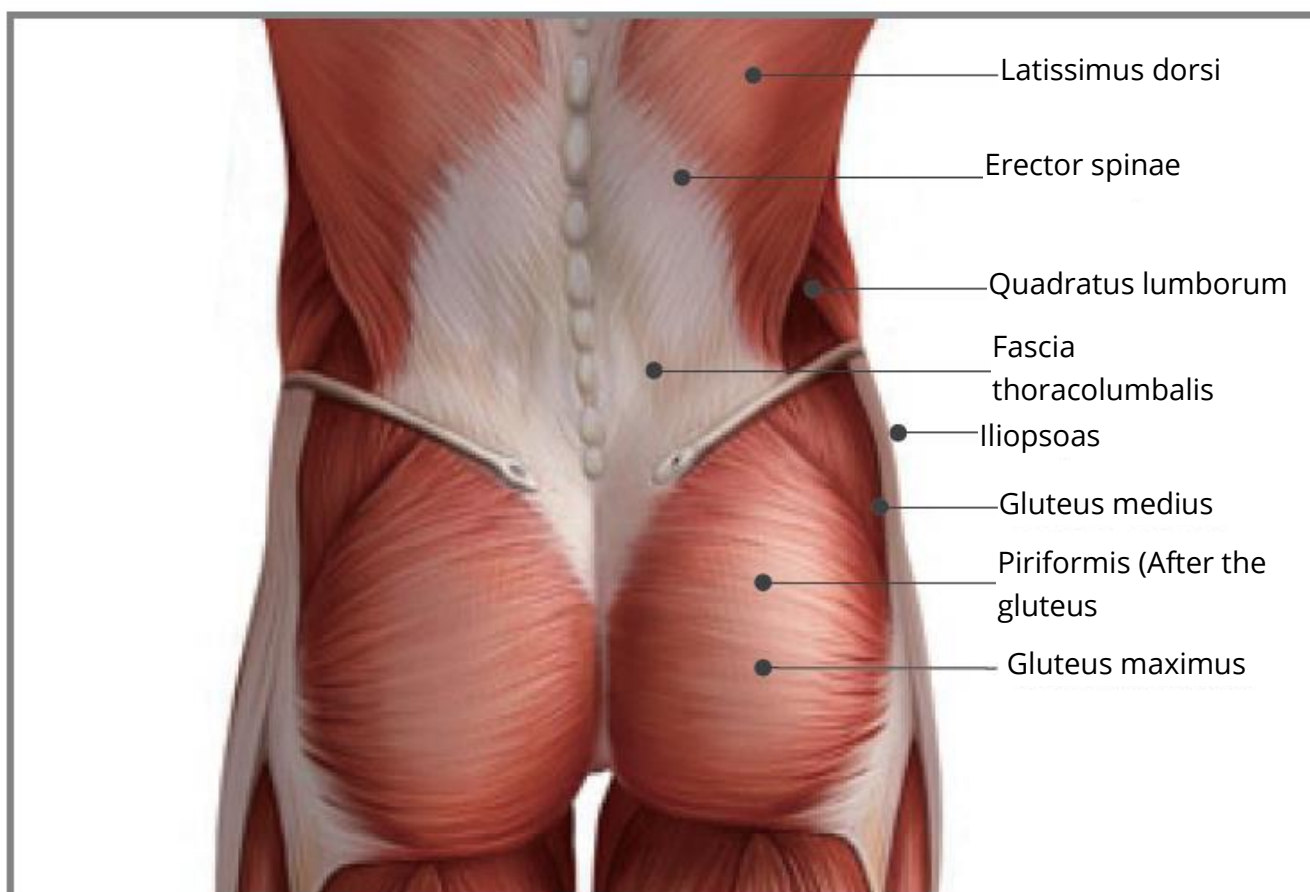
Suggestions: Massage muscles from the middle to both sides and from the top to bottom.

Waist

Position	Duration (s)	Frequency
Erector spinae (lumbar segments)	30 – 60 s	x4 times
Latissimus dorsi	30 – 60 s	x4 times
Thoracolumbar fascia	40 – 60 s	x3 times
Psoas major	30 – 60 s	x3 times
Internal and external oblique muscle of abdomen	10 - 15 s	x2 twice
Erector spinalis (thoracic segments)	15 – 20 s	x2 twice
Quadratus psoas	20 – 30 s	x2 twice
Iliopsoas muscle	20 – 30 s	x3 times
Gluteus maximus	30 – 60 s	x4 times
Gluteus medius	30 – 60 s	x4 times
Piriformis muscle	60 – 90 s	x1 once

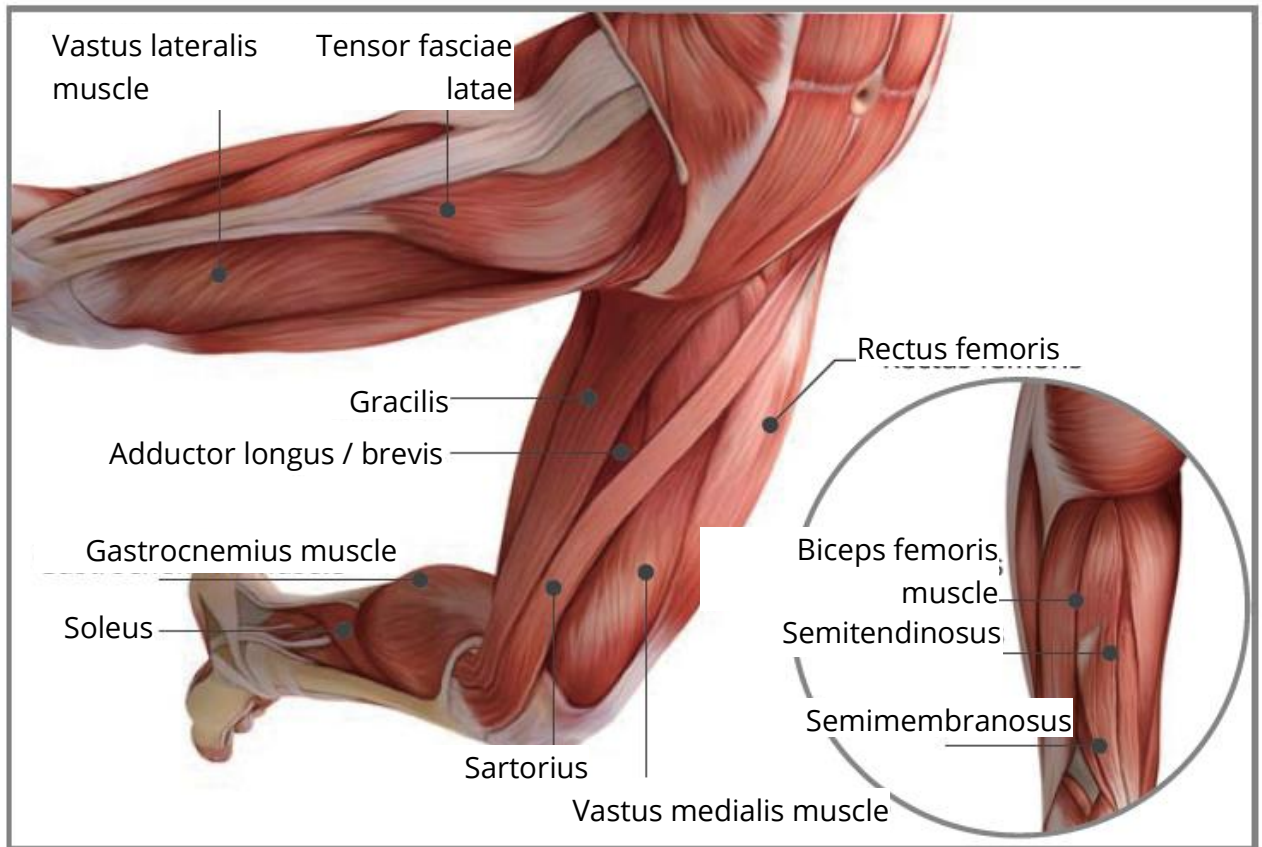
Suggestion: Massage muscles from the middle to both sides and from the top to bottom. Move the device along the muscle when massaging large muscle groups.

Legs

Position	Duration (s)	Frequency
Biceps femoris muscle	20 – 30 s	x3 times
Semitendinosus	20 – 30 s	x3 times
Semi membranous muscle	20 – 30 s	x3 times
Gracilis	10 – 15 s	x2 twice
Adductor brevis	10 – 15 s	x2 twice
Adductor longus	10 – 15 s	x2 twice
Rectus femoris	30 – 45 s	x2 Twice
Vastus latarelis	30 – 45 s	x2 twice
Medial femoral muscle	30 – 45 s	x2 twice
Sartorius muscle	30 – 45 s	X1 once
Tensor fascia lata	15 – 20 s	x2 twice

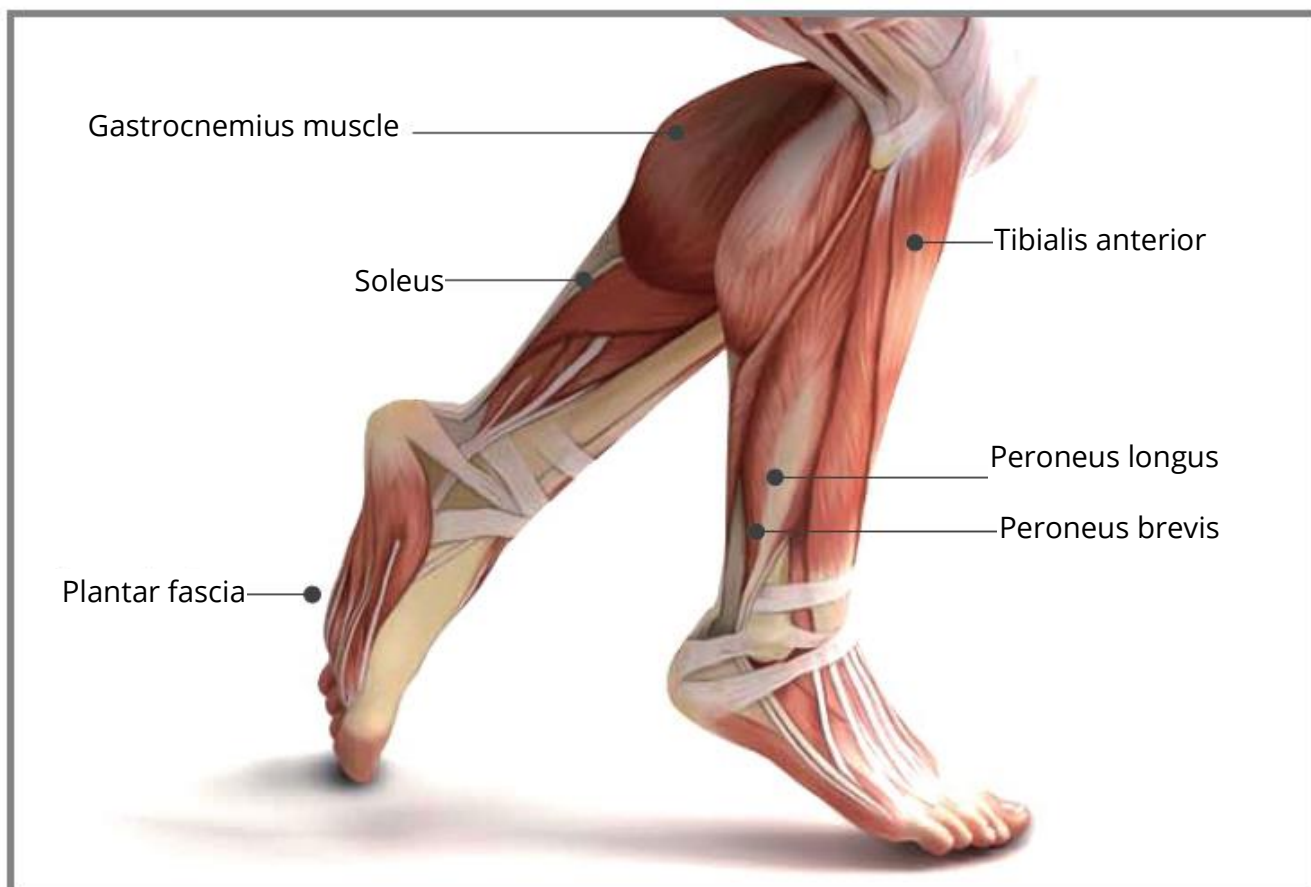Suggestions: Massage muscles from top to bottom.

Foot

Position	Duration (s)	Frequency
Gastrocnemius muscle	20 – 30 s	x3 times
Soleus muscle	20 – 30 s	x3 times
Tibialis anterior muscle	15 – 20 s	x2 twice
Peroneus longus	15 – 20 s	x2 twice
Peroneal brevis	15 – 20 s	x2 twice
Plantar fascia	20 – 30s	x3 times

Suggestions: Massage muscles from top to bottom.

Warranty Conditions

A new product purchased in the Alza.cz sales network is guaranteed for 2 years. If you need repair or other services during the warranty period, contact the product seller directly, you must provide the original proof of purchase with the date of purchase.

The following are considered to be a conflict with the warranty conditions, for which the claimed claim may not be recognized:

- Using the product for any purpose other than that for which the product is intended or failing to follow the instructions for maintenance, operation and service of the product.
- Damage to the product by a natural disaster, the intervention of an unauthorized person or mechanically through the fault of the buyer (e.g. during transport, cleaning by inappropriate means, etc.).
- Natural wear and aging of consumables or components during use (such as batteries, etc.).
- Exposure to adverse external influences, such as sunlight and other radiation or electromagnetic fields, fluid intrusion, object intrusion, mains overvoltage, electrostatic discharge voltage (including lightning), faulty supply or input voltage and inappropriate polarity of this voltage, chemical processes such as used power supplies, etc.
- If anyone has made modifications, modifications, alterations to the design or adaptation to change or extend the functions of the product compared to the purchased design or use of non-original components.

EU Declaration of Conformity

Identification data of the manufacturer's / importer's authorized representative:

Importer: Alza.cz a.s.

Registered office: Jankovcova 1522/53, Holešovice, 170 00 Prague 7

IČO: 27082440

Subject of the declaration:

Title: Massage Gun

Model / Type: Yunmai Slim Elegant

The above product has been tested in accordance with the standard(s) used to demonstrate compliance with the essential requirements laid down in the Directive(s):

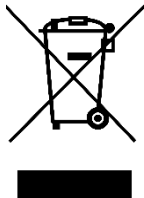
Directive No. (EU) 2014/53/EU

Directive No. (EU) 2014/35/EU

Directive No. (EU) 2011/65/EU as amended 2015/863/EU

WEEE

This product must not be disposed of as normal household waste in accordance with the EU Directive on Waste Electrical and Electronic Equipment (WEEE - 2012/19 / EU). Instead, it shall be returned to the place of purchase or handed over to a public collection point for the recyclable waste. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. Contact your local authority or the nearest collection point for further details. Improper disposal of this type of waste may result in fines in accordance with national regulations.

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