AD-02

Instrukcja obsługi / User Manual /
Bedienungsanleitung / Manuale d'uso /
Manuel de l'Utilisateur / Manual de usuario /
Handleiding / Посібник користувача /
Vartotojo vadovas / Korisnički priručnik

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OVERMAX
You • unlimited

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Introduction

Dear customer!

Thank you for placing trust in us and choosing Overmax.

Thanks to high-quality materials and modern solutions, we provide you with a product perfect for everyday use. We are convinced that the product will satisfy your requirements as it has been manufactured with utmost care. Before you use the product, please read this manual carefully.

Should you have any remarks or questions concerning the product, please do not hesitate to contact us:

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Preface

Before the test

The function of the device is only to indicate the probable alcohol concentration in exhaled air. The test result should not be considered as an undisputed determiner of the ability to safely drive a vehicle, operate a device or perform other hazardous activities.

Because the body's response to alcohol differs tremendously depending on many individual factors, the decision whether one is allowed to perform a given activity cannot be based on the result of the test.

The manufacturer, importer and distributor shall assume no liability for the use of this product in any way. The product cannot be used as a tool to determine whether a person is able to safely or legally drive or operate a motor vehicle or any other device. Even a small quantity of

alcohol has a negative impact on the ability to react and make proper decisions when driving or operating a motor vehicle.

Before you begin testing, please read this user manual carefully and then follow the instructions herein.

General information

This product is a device measuring the concentration of alcohol in the breath. It is equipped with an advanced highly sensitive and durable semiconductor sensor.

Main features

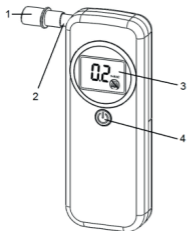
- Advanced alcohol sensor
- Quick response
- SMT assembly;
- smooth and quick operation
- LED indication of test results
- 3-colour LED display
- Stylish design and compact size
- Audio warning beyond the preset level
- Testing error audio warning
- Low energy consumption due to the energy-efficient display
- Portable bag (optional)
- Replaceable mouthpieces

Specifications

- Sensor type: Semiconductor alcohol sensor
- Detection range: 0,00 - 1,58 ‰ BAC
- Alarm trigger level: 0,2 ‰
- Measurement accuracy: $\pm 0,08$ ‰
- Response time: <6s
- Warm-up time: <25s
- Resume Time: <25s

- Supply voltage: DC 3.0V, 2 AAA batteries;
- Optimal usage conditions:
 - Temperature 0°C - 40°C
 - Relative humidity 40% - 90%
- Display: digital LED
- Dimensions: 108 x 40 x 16mm (LxWxH), <45g
- Single charge working time: >200 tests

1. Product illustration and preparation

1. Interchangeable mouthpiece (cap)
2. Folding mouthpiece
3. LED display
4. Power button

Preparation

1. Install the batteries: Slide the rear cover down.. Install 2 AAA alkaline batteries 1.5V. Slide the cover back in place.
2. After the batteries are installed the device starts the initialization. During the process the „INITIALIZE” message flash on the screen for 1 minute. When the initialization is completed the breathalyzer can start a test.

Rear cover

**AAA1.5V alkaline
batteries**

**Folded
mouthpiece**

**Unfolded
mouthpiece**

2. Test instructions

2.1. Use the Power button to turn the breathalyzer on.

2.2. Breathalyzer turns on and emits a short beep. The „PH” message is shown on the screen for a brief moment. Then, it is displaced by a countdown timer that shows the test preparation time. Please wait. After 20 seconds the device beeps and the screen shows „BLOW” message. Now you can start the test.

2.3. The first step is to take a deep breath, then blow directly into the mouthpiece's hole until you hear a beep sound.

2.4. When the test is completed the device beeps, and the result is displayed on the LCD screen. The alarm is off as long as the concentration of alcohol in breath is lower than the present level. However, when it is exceeded, the device emits long beep sound and the red warning LED turns on. The test result is displayed on the screen for 35 seconds, then the device turns off.

2.5. If you need to do the test again, please repeat the steps described in sections 2.2 - 2.4.

3. Usage notes

- 3.1.** Do not allow the device to be dropped or exposed to shock.
- 3.2.** The device may not function properly in places with a high concentration of gases.
- 3.3.** The tests performed at low-battery condition are of reduced measurement accuracy, which increases the risk of incorrect results, different from the actual values.
- 3.4.** To prevent measurement errors and reduce the risk of incorrect results, when you stop drinking, please wait 10 minutes before starting the test.
- 3.5.** When operating the device please strictly follow the instructions indicated in this manual. If the device has not been used for a long time, the results of the first two tests may differ from the actual values. Thus, the third measurement should be considered as the most accurate one.
- 3.6.** The product should not be used or stored in places where it would be exposed to chlorine or other corrosive gases.
- 3.7.** Under normal circumstances, the life cycle of the sensor can exceed 2 years.
- 3.8.** To clean the breathalyzer, wipe it with a clean cloth. Do not use any hard objects or solvents containing corrosive substances.

The pictures are for reference only. The real design of the products may differ from the pictures presented.

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