AngelSounds JPD-100S Mini

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Product Introduction


The AngelSounds JPD-100S Mini is a lightweight and portable detector for listening to the heartbeats of a child's fetus.

Note: For proper functioning, the fetus must be at least 12-14 weeks old, ideally 16 weeks. Consult your doctor about the use of the device.

Package Contents

AngelSounds JPD-100S Mini, battery, headphones, manual.

Basics of use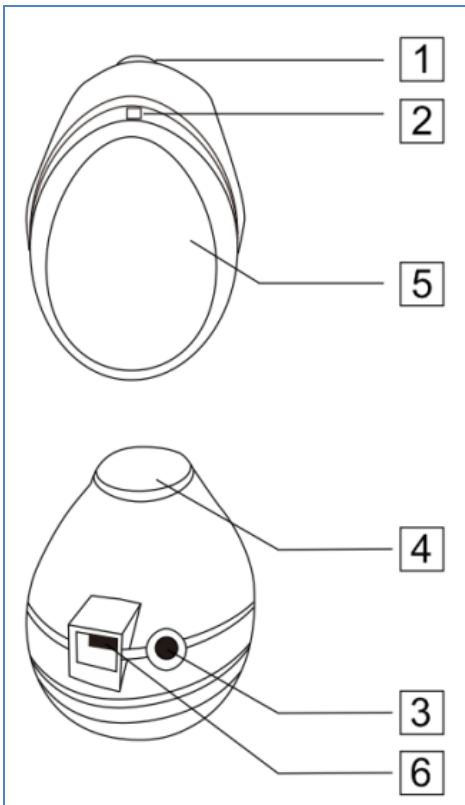

1. Power on/off, adjust volume

2. Power indicator

3. Headphone jack

4. Signal transmission

5. Battery cover


6. Audio signal jack

Turn the wheel (1) on the device, the LED lights up to turn on the device. Gradually turning the wheel adds volume, turning to the opposite side gradually reduces the sound until the device is completely turned off.

The device offers two connectors. Headphone jack and audio jack. You can use this by connecting the recording cable (not included) to record the sound of the fetal heartbeat. If you don't need to

record audio, just connect the supplied headphones to listen.

Battery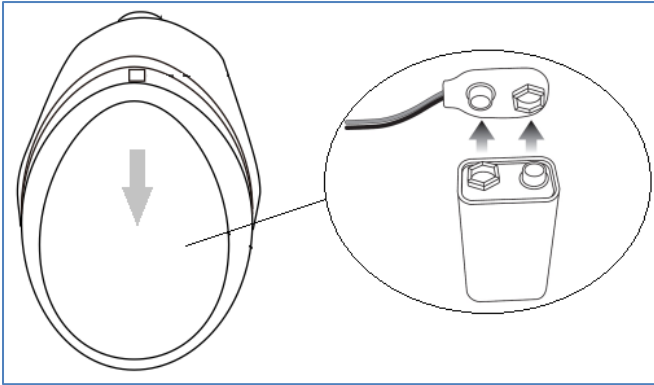


Battery included. It's located directly on the device, but not connected. Before first use, the battery must be connected as shown in the following picture.

Remove the cover (5) and connect the

battery.

How to find the heart of the fetus

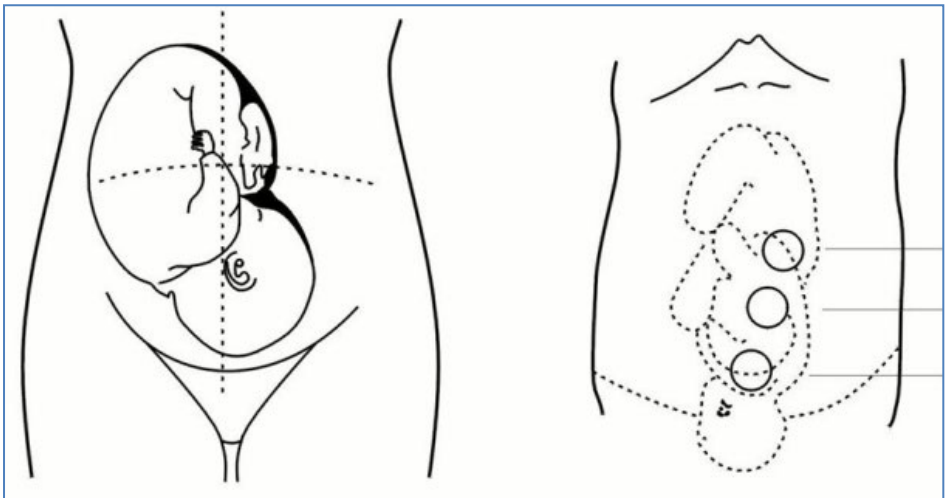Apply a little gel to the signal transmission site (not included) and place the device on your stomach. Move it slowly, trying to find the heart of the fetus. After finding the sound, continue to move the probe slowly, looking for the place where the sound signal will be clearest.

The usual position of the fetus can be seen in the following picture.

However, the position of the fetus may vary depending on the stage of pregnancy and other conditions.

Count the number of sounds per minute and get information about your heart rhythm. We recommend that you repeat the counting more than once.

Audio recording

You can also record these sounds while listening to heart sounds. Just connect the recording cable (not included) to the miniUSB connector, plug the other end into the device on which the sound will be recorded and start recording.

Safety Warnings

Avoid using the device in an environment with a potential risk of explosion, high or very low temperature or humidity, near an open flame. We recommend that you do not use the device outdoors. Always wipe off the gel from the device after monitoring. Do not use the device if it shows signs of damage! If you clean your device, always keep it turned off

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