R1

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Dear customer,

Thank you for purchasing our product. Please read the following instructions carefully before first use and keep this user manual for future reference. Pay particular attention to the safety instructions. If you have any questions or comments about the device, please visit our customer support page: www.alza.cz/EN/kontakt.

App Installation


Do not use the device unless it is successfully connected to your mobile phone via the Android or iOS app and you are streaming live video.

To download the app, search for "Bebird" in your phone's App store or Google Play. Alternatively, scan the QR code to download and install the Bebird app.


App Connection

- Open the app to connect the WiFi in the settings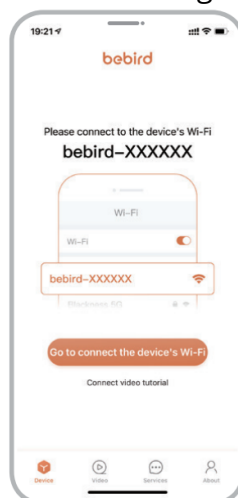

- Press the power button to turn on the device. A flashing blue light indicates that the device is powered on.
- Connect to the device's WiFi 'bebird-R1-XXXXXX'.
- Return to the app and begin using the device.

Accessory Replacement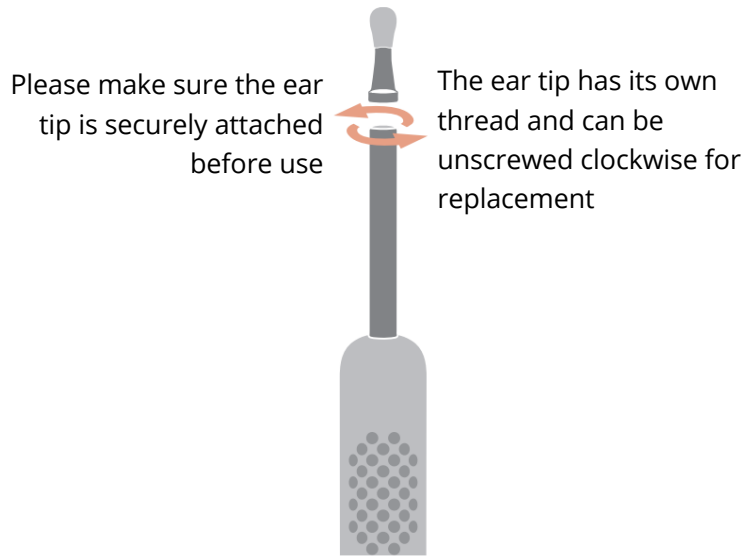

For a better user experience, we recommend replacing the ear tip at least every three months. This interval may be adjusted depending on how frequently the device is used.

Accessories


Model bebird R1+

PC ear spoon x 1

Cotton swab ear tip x 1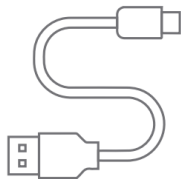

Charging cable x 1

Storage tube x 1

Model bebird R1+S

Metal and silicone ear spoon x 1


Cotton swab ear tip x 2

PC ear spoon x 3

Storage tube x 1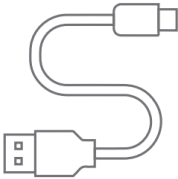

Charging cable x 1

Storage box x 1

Model Name bebird R1+Plus


Metal and silicone ear spoon x 1


Cotton swab ear tip x 3

PC ear spoon x 5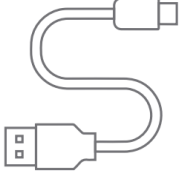

Charging cable x 1

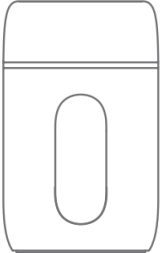Standing base x 1

Note: Do not use the charging equipment with output voltage exceeding 5V.

Troubleshooting

Problem	Solution
No response after startup	Use the enclosed charging cable to charge. The charging time is 50 minutes.
The app closes quickly	Delete the existing app and reinstall it
Difficulty replacing the ear tips	Align the replacement ear tip with the camera and tighten it clockwise.
The indicator light blinks quickly	Use the enclosed charging cable to charge. The charging time is 50 minutes.
Abnormal display in app	Interrupt the connection, restart the device to connect again

Product Parameters

Product Name	Smart visual ear cleaner
Pixel	8 Megapixels
Depth	10-50 millimetres
Lens diameter	3.5 millimetres
Image transmission rate	20fps
Battery	180mAh lithium battery
Battery life	70 minutes
Charging time	50 minutes
Input current	DC 5V=0.5A
Gyroscope	6 axis
Network standard	IEEE 802.11b/g
Working frequency	2.4GHz
Rod weight	13 grams
Image sensor	CMOS
Working temperature	-10 to 50°C
Optimum focal length	13.5 millimeters
Maximum wireless output	<20dBm

Attention

- When cleaning the lens, wipe it carefully with professional alcohol cotton swabs.
- Please observe your surroundings before use. Stay at least 0.5 meters away from people, animals, or moving objects. Do not use in a moving vehicle or other unstable environments. Remain stationary while using.
- This product is unsuitable for children under 3 years old.
- This product is not suitable for children to use alone.
- Do not place the product in a humid environment for long periods or soak it in liquids.
- The product has a built-in rechargeable lithium-ion battery. Even if not in use, charge it once a month to prevent long-term battery damage.
- Use a 5V charger to charge the device. Do not use the product while it is charging.
- Avoid direct exposure to sunlight.
- Do not place heavy objects on the product.
- It is normal for the device temperature to rise slightly during use (up to 35°C).

Warranty Conditions

Warranty policies may vary by country or region. For specific details such as the warranty period and coverage, please refer to the after-sales policy provided by the sales channel from which you purchased the product. If you need to request after-sales service, please contact the sales channel or authorized dealer where the product was purchased. Be sure to provide valid proof of purchase, such as an invoice or order number, to confirm your eligibility for service.

EU Declaration of Conformity

This product complies with the legal requirements of the following European Union directive: 2014/53/EU.

WEEE

This product must not be disposed of as normal household waste in accordance with the EU Directive on Waste Electrical and Electronic Equipment (WEEE - 2012/19/EU). Instead, it shall be returned to the place of purchase or handed over to a public collection point for the recyclable waste. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. Contact your local authority or the nearest collection point for further details. Improper disposal of this type of waste may result in fines in accordance with national regulations.

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