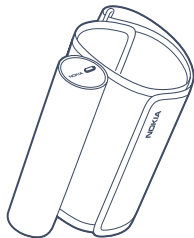
# Nokia BPM+

## QUICK INSTALLATION GUIDE

---

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This guide is available in additional languages at: [www.nokia.com/health/guides](http://www.nokia.com/health/guides)

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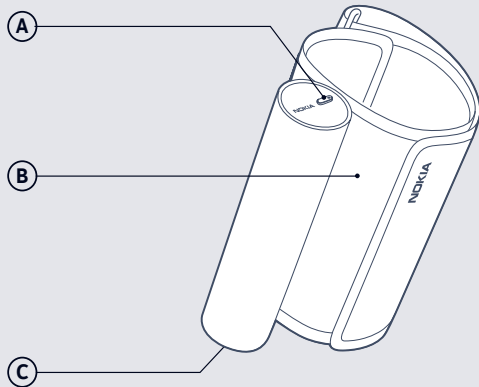
---

1

**Visit [healthapp.nokia.com](https://healthapp.nokia.com) from your smartphone or tablet, and follow the instructions to install the Health Mate™ app.**

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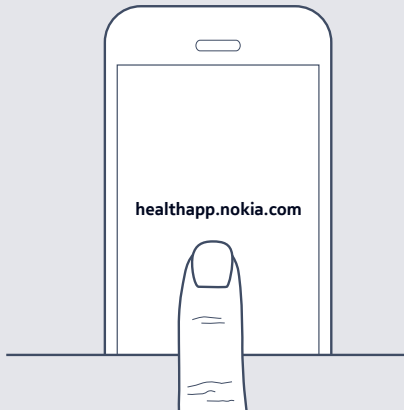
## 2

**In the app, follow the instructions to install Nokia BPM+.**

- PL** W aplikacji postępuj zgodnie z instrukcjami, by zainstalować Nokia BPM+.
- CS** V aplikaci postupujte dle pokynů pro instalaci aplikace pro výrobek Nokia BPM+.
- HU** Az alkalmazásban kövesse az utasításokat a Nokia BPM+ berendezés telepítéséhez.
- SK** V aplikácii postupujte podľa pokynov a nainštalujte Nokia BPM+.

# How to take a blood pressure measurement

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**SK** Posad'te sa do pohodlnej polohy.

## 2

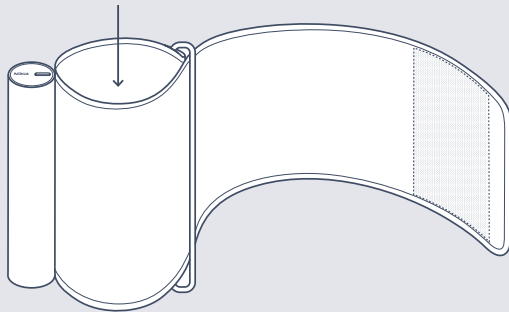
**Unroll cuff. Place your arm inside it.**

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3

**Tighten the cuff around your arm. The metal tube should be positioned against your inner arm.**

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4

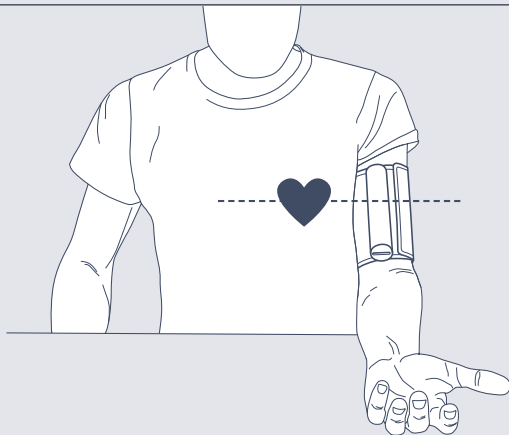
**Place your arm on a table and level with your heart.**

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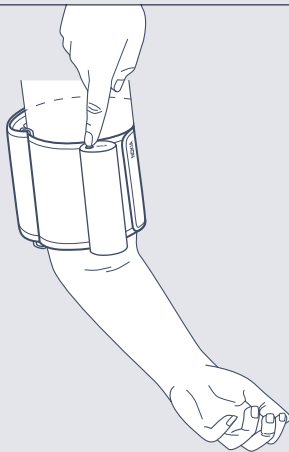
---

5

**Turn on your blood pressure monitor.  
The app starts automatically.**

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## 6

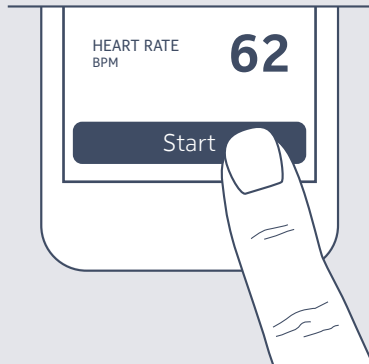
**From the app, press the start button launch the measurement.**

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12

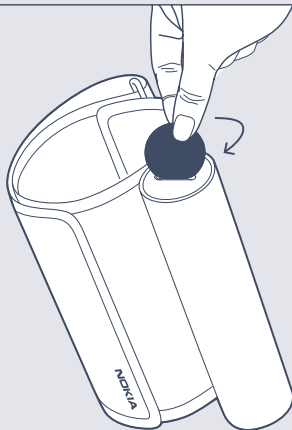
**Use a coin to remove the battery cover.**

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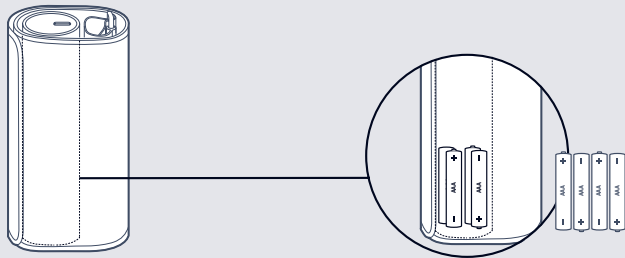
CS Monitor krevního tlaku

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SK Tlakomer

## AAA alkaline cells (already inserted, X4)

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### EN This device complies with Part 15 of the FCC Rules and with Industry Canada licence-exempt RSS standard(s).

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications made to this equipment not expressly approved by Nokia may void the FCC authorization to operate this equipment.

#### Radiofrequency radiation exposure information :

The radiated output power of the device is far below the FCC radio frequency exposure limits. Nevertheless, the device shall be used in such a manner that the potential for human contact during normal operation is minimized.

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Maximum output power :

4.45 dBm

Withings (Part of Nokia) hereby declares that the device Nokia BPM+ is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU.

The declaration of conformity can be found at: [nokia.com/health/compliance](http://nokia.com/health/compliance).

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➔ [nokia.com/health/support/bpmplus](https://nokia.com/health/support/bpmplus)

Nokia BPM+ | Compact Wireless Blood Pressure Monitor | v1.0

© 2017 Nokia Technologies Ltd. All rights reserved.