en

**HYGIENE**

Don't touch the funnel. It is mandatory to keep distance of 3 cm of the object or the person to be tested. A test person's skin, any secretions or surfaces must not be touched with the funnel. Also avoid the direct contact with substances to be tested. In case of an contamination change the funnel.

**AVOID**

Avoid taking the samples within the draft of wind or air in rooms and open areas. The test result could be affected.

**CAUTION**

Damage to the device! ► Do not charge alkaline batteries, as the device may be damaged otherwise.

**CAUTION**

Danger of explosion. ► Do not expose to fire ore force to open.

**CAUTION**

Damage to the instrument. Leakage of battery acid can cause damage to the device. ► Remove batteries if storing the device longer than 6 month.

**CAUTION**

Damage to the instrument. ► Do not submerge the device in liquid. ► Do not use cleaning agents containing alcohol, lemon, or solvents.

Batteries must not be disposed of as household waste. This is indicated by the adjacent symbol. Dispose of batteries at battery collection centres as specified by the applicable regulations.

This product must not be disposed of as household waste. This is indicated by the adjacent symbol. You can return this product to Dräger free of charge. For information please contact the national marketing organisations or Dräger.

Ambient conditions

During operation

-15 °C to +50 °C / +5 °F to +122 °F | 10 to 100 % rel. hum. (non-condensing) 600 to 1300 hPa

During storage

-20 °C to +70 °C / -4 °F to +158 °F

USB communication socket for service only.

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