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Soft&Clean, Sicherheits-Nasensauger
Art.Nr.: 79112

DE

Instruction manual

Soft&Clean, Safety nasal aspirator
Art.no.: 79112

GB

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Please read the instructions carefully prior to applying this device. Otherwise, the safety functions of the item could be impaired. Please keep the instructions in a safe place for future reference

Attention! This article has been developed for the benefit of your children. It is not a toy and no substitute for supervision by adults.

Safety instructions: Use the item for its intended purpose only. Do not use the product if there is a sign of damage. This product is not a toy! Store away from children.

Application: Press the pump ball in the middle, and carefully insert the suction tube in the blocked nostril. Gently close the other nostril with the finger. Then reduce the pressure on the pump ball and suction the secretions out.

Cleaning and storage: Clean the nasal aspirator before the first use with a mild cleaning agent and warm water. Please do not use any chemicals that could damage the material. Disinfection in a steriliser or by boiling in water for 5 min.

Please retain your receipt, the user manual and part number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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