Digital Display Alcohol Concentration Test Meter

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The product user guide contains product features, how to use, and the operating procedure. Read the user manual carefully to get the best experience and avoid unnecessary damage. Keep this manual for future reference. If you have any questions or comments about the device, please contact the customer line.

✉ www.alza.co.uk/kontakt

☎ +44 (0)203 514 4411

Importer Alza.cz a.s., Jankovcova 1522/53, Holešovice, 170 00 Praha 7, www.alza.cz

Precautions

- Thank you for selecting and purchasing digital display alcohol concentration test meter.
- Please read this manual carefully to ensure that you can use the product correctly.
- This device is a healthcare product; it is intended to control overdrink of alcohol so as to keep you fit, through testing the alcohol in your breath and calculating the content of alcohol in the blood via microcomputer. The product is applicable to family healthcare as well as for reference in determining whether the driver is driving after drinking.
- The test result may be affected by a number of aspects such as environmental temperature, various gases in the air and smoking, etc. the result shall, therefore, be only for reference and shall not be regarded as the justification of driving after drinking.
- No matter whether you have consumed this product, its manufacturer and appropriate agents shall be not responsible for any traffic accident arisen due to ethanol or other accidents happened under the related regulations and rules.
- As people's breath contains some fermentable gases (from stomach and intestines), so some gases may even be tested out under no drink circumstance.
- When after drinking, it is recommended to last after 20 minutes, it is because 20 minutes is approximately for alcohol to be absorbed into blood from the digestive organs, and residual alcohol remaining in the mouth takes this long to dissipate.
- For accurate result, do not repeat neat testing within 3 minutes after time.
- Do not smoke and eat for 20 minutes prior testing.
- Do not use any excitant substances near the tester, such as paint, insecticides, alcohols.
- If the device has been performed for five times successively, please stop for two minutes and then use it again.
- Do not drive after drinking!

Operation

Wait 20 minutes	No Smoking	No Eating	No Drinking
			

BAC	Dose-Specific Effects
0.020-0.039%	Generally, no loss of coordination. Slight joy or loss of shyness. Depressant effects are not clear.
0.040-0.069%	General feelings of well-being, relaxation, shyness is reduced and sensation of warmth. Joy. Some minor impairment of reasoning and memory. Lowering of caution. Driving skills may be impaired.
0.070-0.099%	Slight impairment of balance, speech, vision, reaction time, and hearing. Joy. Judgment and self-control are reduced. Caution, reason, and memory are impaired. Driving skills are always impaired.
0.100-0.129%	Major impairment of motor coordination and loss of good judgment. Speech may be slurred. Balance, vision, reaction time and hearing will be impaired. Joy.
0.130-0.159%	Gross motor impairment and lack of physical control. Blurred vision and major loss of balance. Joy reduced and state of feeling unwell begins to appear.
0.160-0.199%	Anxiety, restlessness predominates. Nausea may appear. Appearance of a "sloppy drunk."

Preparation before using


- After opening the package, please check the parts and components below. If anyone is absent, please contact its dealer.
- Accessories

1. One digital display alcohol concentration detector.
2. One User Manual.
3. Two AA alkaline batteries. (Not included).
4. 5 air-blowing pipes.
5. One bag.

Package Contents

Components Diagram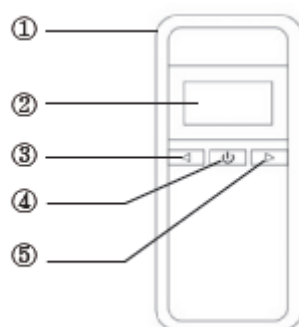

1. Mouthpiece Connector

2. LCD Display

3. Left key (K1)

4. Power Button On/Off

5. Right Key (K2)

Installing Battery

1. Open the back cover

2. Install two 1.5V AA alkaline batteries or two batteries with same specification into the battery case in a right way (as shown in figure 1) per the mark “+” and “-” on the case, and then close the cover well (as shown in figure 2)

3. Install the back cover

- Battery Replacement

If the Battery symbol on the LCD flashes, it indicates the battery electricity is not enough and the batteries need to be replaced, in order to ensure the test accuracy. Open the battery cover and take out the batteries and reinstall the new batteries with same specification (as shown in figure). Make sure to keep the old batteries out of children's reach, please dispose the useless batteries according to the local regulations.

1. Battery Compartment, 2. Space for mouthpieces

Set temperature display format


Press the power button about 2 seconds, the alcohol tester LCD will turn on with one beep sound. Figure (1)

Press right k2 to set the temperature unit conversion.

How to use the alcohol tester


- Operational Procedure:
- Wait
- Press the power button about 2 seconds, the alcohol tester LCD will turn on with one beep sound.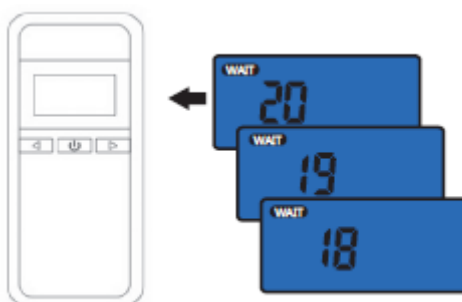


- The: "WAIT" symbol is displayed and the 20 to 0 countdown is started that indicate the tester is in work up mode.
- NOTE
- If first work up is defeated, the tester will be reset automatically and work up again for 1-2 times.
- It always has a large number of gas and impurity adsorbed on the surface of alcohol sensor, when first time taking out the tester from the packaging, or after long time no using for the tester, it needs more time for cleaning and work up the alcohol sensor, you should repress the power button and retry several times if possible.

Test

- The “RDY” symbolises displayed when work-up is succeed, and the countdown is started that indicate the tester is in testing mode.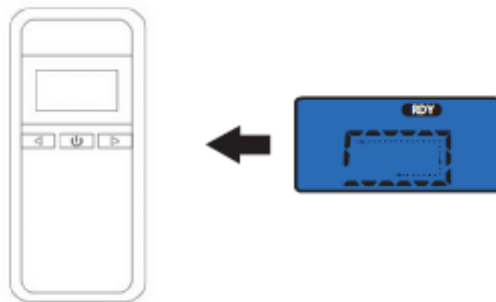

- Near and blow into the breath inhale for 3-5 seconds.
- The analysis of...

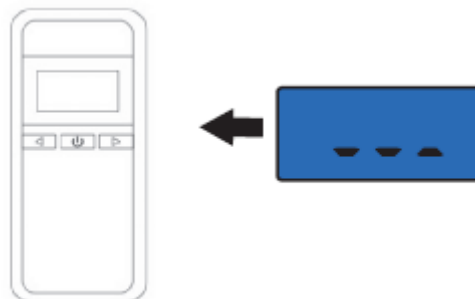Read the tester result from the alcohol tester lcd.

Caution $\leq 0.200\% \text{BAC}$

$0.210\% \text{BAC} \leq$ Caution $\leq 0.490\% \text{BAC}$

Danger $\geq 0.500\% \text{BAC}$

Unit Conversion

Auto Power Off

1. After the result is displayed for about 10 seconds, the "off" symbol will appear 2 seconds and the alcohol tester will be turned off.
2. If warm up is defeated, the tester will be automatically turned off.
3. Long press the POWER button to turn on the product and long press it again to turn off in any mode.

Restart Mode

After displaying the testing result, click POWER button to re-activate the TESTING mode directly, no need to wait till it shut off to click the POWER button.

Set unit Result display Format

Explanation for the 4 kinds of units in the sheet.

Value	Meaning
0	%BAC
1	‰BAC
2	Mg/L
3	g/l

Setting Method

1. In the TESTING mode after turning on the product, long press left button (K1) within the 4 seconds of displaying "WAIT", so as to go into the POINTSETTING mode. Please check the following picture.

2. Then click the left button and Right button (K1) (K2) to select the unit you need, after setting up the unit, long press right button of K1 to quit the UNIT SETTING mode and return back to TESTING mode.

3. If there is not any operation within 15 seconds it will show "OFF" to shut off the product.

Set alert level

Setting Method

1. Double click the POWER button in UNIT SETTING mode to go into CAUTION setting mode, as shown the following picture.
2. Set up CAUTION first, then click left button (K1) and right button (K2) to set up CAUTION data.
3. After setting up the CAUTION data, single click POWER button to switch to DANGER setting, then click the left and right button of K1 K2 to set up the CAUTION data.
4. After finish setting, double click POWER button to quit the CAUTION mode, and return back to UNIT SETTING mode.

Caution $\leq 0.200\% \text{BAC}$

$0.210\% \text{BAC} \leq$ Caution $\leq 0.490\% \text{BAC}$

Danger $\geq 0.500\% \text{BAC}$

REC record checking mode

1. Within the TESTING mode time, double click POWER button to get into the REC record checking mode, click the left and right button of K1 K2 to check the 1-32 times TESTING record.
2. In REC mode, double click POWER button to quit the checking mode and return back to TESTING mode.
3. If there is not any operation within 15 seconds it will show "OFF" to shut off the product.

Declaration

1.Regardless of whether the product is used, manufacturers and vendors will not bear any responsibility for the violations. For example: drunk driving and high-altitude operations driving.

2.Each person reacts differently to alcohol, so the test results are for reference only, which cannot serve as the criteria in determining the outcome.

Common Faults and Solutions

Faults	Possible Faults Reason	Solution
Power on failing	Put batteries incorrectly	Installing batteries in correct polarity
	Very low power batteries	Replacing the new alkaline batteries
	Circuit Fault	Contacting aftersales service
Blowing no response	Warm up not completing	Warm up for more times
	Circuit Fault	Contacting aftersales service
Blowing no result	Residual Gas in tester	Warm up again
	Not blowing in schedule time	Blowing during "BLOW" countdown time
Now Blowing with test result	Residual gas in tester	"Warm up" for more times
Different result after several testings	Testing as soon as drinking	Testing after 20 minutes of drinking
	Blowing time different	Deep breath for 3 – 5 seconds

Different country alcohol units

No.	BAC (Blood Alcohol Concentration)				BrAC (Breath Alcohol concentration)	
	%	‰	g/l	mg/100ml	mg/l	ppm
1	0,01	0,1	0,1	10	0,05	25
2	0,02	0,2	0,2	20	0,10	50
3	0,03	0,3	0,3	30	0,15	75
4	0,04	0,4	0,4	40	0,20	100
5	0,05	0,5	0,5	50	0,25	125
6	0,06	0,6	0,6	60	0,30	150
7	0,07	0,7	0,7	70	0,35	175
8	0,08	0,8	0,8	80	0,40	200
9	0,09	0,9	0,9	90	0,45	225
10	0,10	1,0	1,0	100	0,50	250
11	0,11	1,1	1,1	110	0,55	275
12	0,12	1,2	1,2	120	0,60	300
13	0,13	1,3	1,3	130	0,65	325
14	0,14	1,4	1,4	140	0,70	350
15	0,15	1,5	1,5	150	0,75	375
16	0,16	1,6	1,6	160	0,80	400
17	0,17	1,7	1,7	170	0,85	425
18	0,18	1,8	1,8	180	0,90	450
19	0,19	1,9	1,9	190	0,95	475
20	0,20	2,0	2,0	200	1,00	500

Common Units:

%BAC – USA

g/l – European

mg/L BRAC – Japan

mg/100ml – China

‰BAC – Russia

‰prom – Poland

Maintenance

- Do not use these cleansers such as benzene, thinner or petrol to clean the product to avoid damaging its surface.
- Do not fall it down or strike, otherwise malfunction may be caused.
- Do not try to repair, rebuild, or dismantle any parts except for the battery replacement.
- When not use the device for a long time, you are recommended to take out the batteries to prevent the damage due to battery leakage.

Performance Parameter

- Measure range:
- 0.000-0.199%BAC.
- 0.000-1.999‰BAC
- 0.000-1.999g/l.
- 0.000-0.995mg/l
- Alarming Level: 0.05%BAC; 0.5‰BAC; 0.5g/l; 0.25mg
- Accuracy: $\pm 0.01\%$ BAC; $\pm 0.1\%$ ‰BAC; ± 0.05 mg/l
- Figure size: Length 114mm
- Width 51mm
- Height 18mm
- weight: 60g
- Battery: 2×1.5V (AA battery)
- blowing Time; 5s
- Display: LCD Display with backlight
- Current consumption: working current ≤ 180 mA
- Quiescent Current $\leq 40\mu$ A
- Environmental temperature: -10°C~50°C
- Environmental humidity: $\leq 95\%$ NO Dews
- Battery expectancy time: ≥ 300

Warranty Conditions

A new product purchased in the Alza.cz sales network is guaranteed for 2 years. If you need repair or other services during the warranty period, contact the product seller directly, you must provide the original proof of purchase with the date of purchase.

The following are considered to be a conflict with the warranty conditions, for which the claimed claim may not be recognized:

- Using the product for any purpose other than that for which the product is intended or failing to follow the instructions for maintenance, operation and service of the product.
- Damage to the product by a natural disaster, the intervention of an unauthorized person or mechanically through the fault of the buyer (e.g. during transport, cleaning by inappropriate means, etc.).
- Natural wear and aging of consumables or components during use (such as batteries, etc.).
- Exposure to adverse external influences, such as sunlight and other radiation or electromagnetic fields, fluid intrusion, object intrusion, mains overvoltage, electrostatic discharge voltage (including lightning), faulty supply or input voltage and inappropriate polarity of this voltage, chemical processes such as used power supplies, etc.
- If anyone has made modifications, modifications, alterations to the design or adaptation to change or extend the functions of the product compared to the purchased design or use of non-original components.

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